MODEL #: **TP-5555**





Rx only


Triple Play VT[®] – Universal Shoulder Orthosis APPLICATION INSTRUCTIONS

DESCRIPTION: Shoulder orthosis, acromio/clavicular (canvas and webbing type,) prefabricated. The orthosis has two (2) chambers that fill from the distal upward toward the proximal end of the extremity.

1. Remove optional gel pack and place in freezer for a minimum of 1 hour.
NOTE: Each time the gel wrap thaws, refreeze for 1 hour.
2. Remove gel pack from freezer and re-attach to the inside of the wrap. Make sure to keep a protective thin cloth barrier between the skin and gel pack.
3. Thread arm through lower loop, and pull wrap up onto shoulder. Secure the middle strap around arm, and tighten lower and middle straps to achieve a snug fit. Take the longer strap and thread through the back clip so that it attaches across the back, under the unaffected arm and across the chest. Thread through clip and secure with tabs.
4. Once the wrap is on, secure the optional immobilizing sling to the front portion of the wrap. Place your wrist through the loop so that it comfortably approximates the arm at a 90° angle.
5. Refer to the QuickStart Guide or Instructions for Use included in the Triple Play VT packaging for instructions on operating the unit with orthotic wrap.
NOTE: Ensure that the valve switch on the wrap is in the ON position (vertical) before powering on TPVT unit.
6. Once the pressure is reached, you may choose to hold the pressure by closing the valve on the connector by turning the valve lever perpendicular (or 90°) to the hose line. If desired, the hose can be disconnected from the wrap by twisting the disconnect valve connector to the left.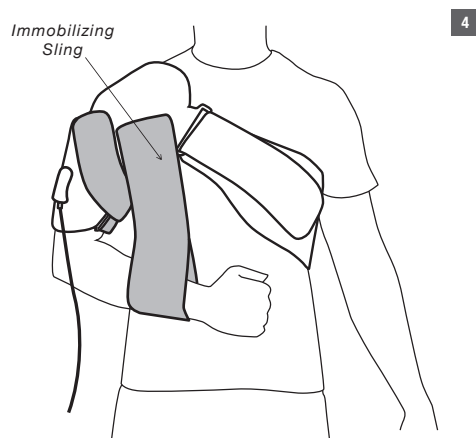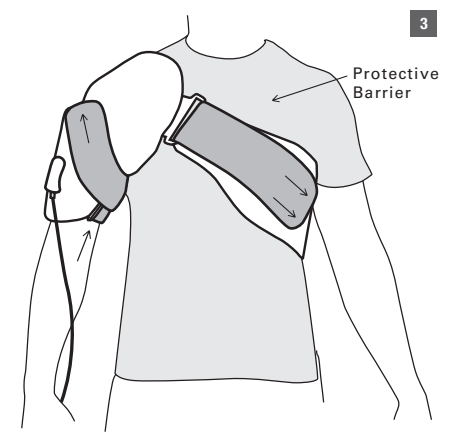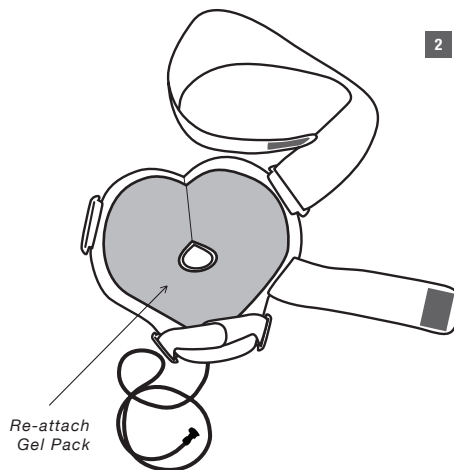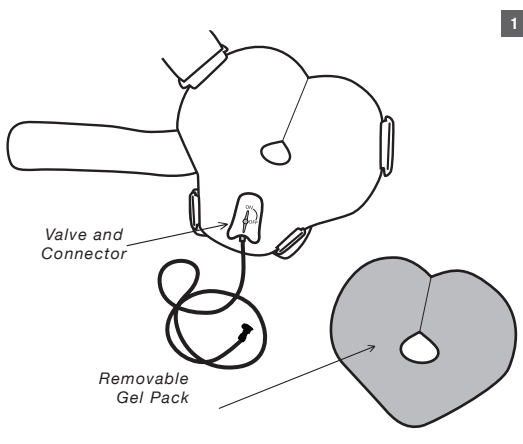


Valve Operation

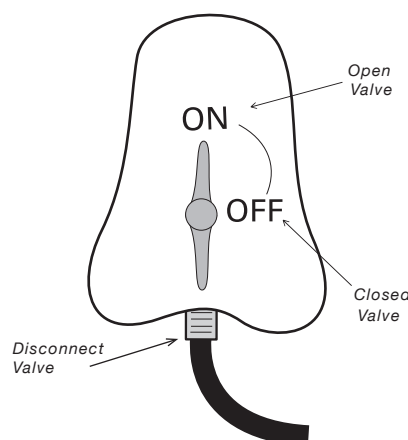OPTIONAL HAND BULB

Your product is provided with an optional hand bulb which can be used to inflate/deflate the wrap in the absence of the Triple Play VT unit. If you desire, you may use scissors to cut the tubing just below the insertion into the wrap. Then place the barbed connector on the bulb into the tubing connected to the wrap.

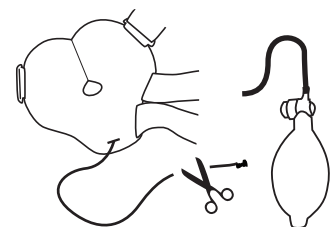For questions about this product call: **800.994.0464**

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