MODEL #: **TP-4444**

# Triple Play VT<sup>®</sup> – Universal Knee Orthosis APPLICATION INSTRUCTIONS

**DESCRIPTION:** Knee orthosis, elastic with condylar pads and joints, with or without patellar control, prefabricated. The orthosis contains rigid metal stays or stabilizers to support the weakened body member. The orthosis has two (2) chambers that fill from the distal upward toward the proximal end of the extremity.

1. Remove optional gel pack and place in freezer for a minimum of 1 hour.

*NOTE: Each time the gel wrap thaws, refreeze for 1 hour.*

2. Remove the adjustable hinges from the inside of the brace. Secure these hinges back down to the brace in a location that will allow the condyle pads to rest securely on the inside and outside of the knee.

*NOTE: For smaller knees, these hinges will be located closer to the center, for larger knees these hinges will be located closer to the edges.*



3. Remove gel pack from freezer and secure to the inside of the brace making sure to not place it over the condyle pads.

4. Place a protective barrier between the gel pack and the skin. A

cloth sleeve is included in your package. Any thin cloth barrier will be effective. Place the brace over the knee and secure the two straps around the knee to achieve a snug fit.

5. Refer to the QuickStart Guide or Instructions for Use included in the Triple Play VT packaging for instructions on operating the unit with orthotic wrap.

7. Once the pressure is reached, you may choose to hold the pressure by closing the valve on the connector by turning the valve lever perpendicular (or 90°) to the hose line. If desired, the hose can be disconnected from the wrap by twisting the disconnect valve connector to the left.

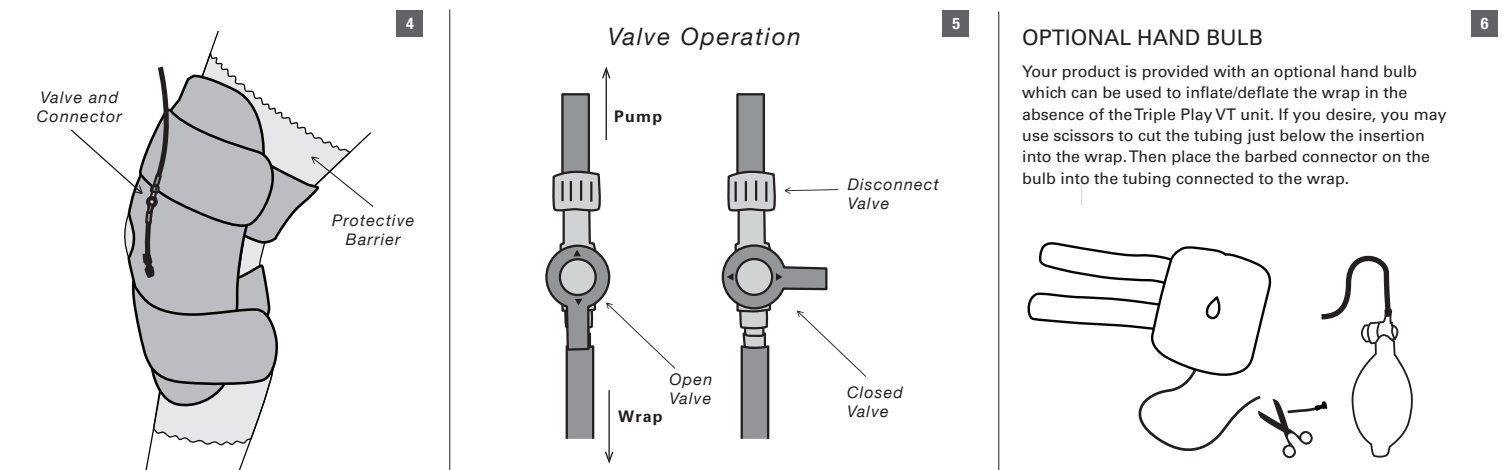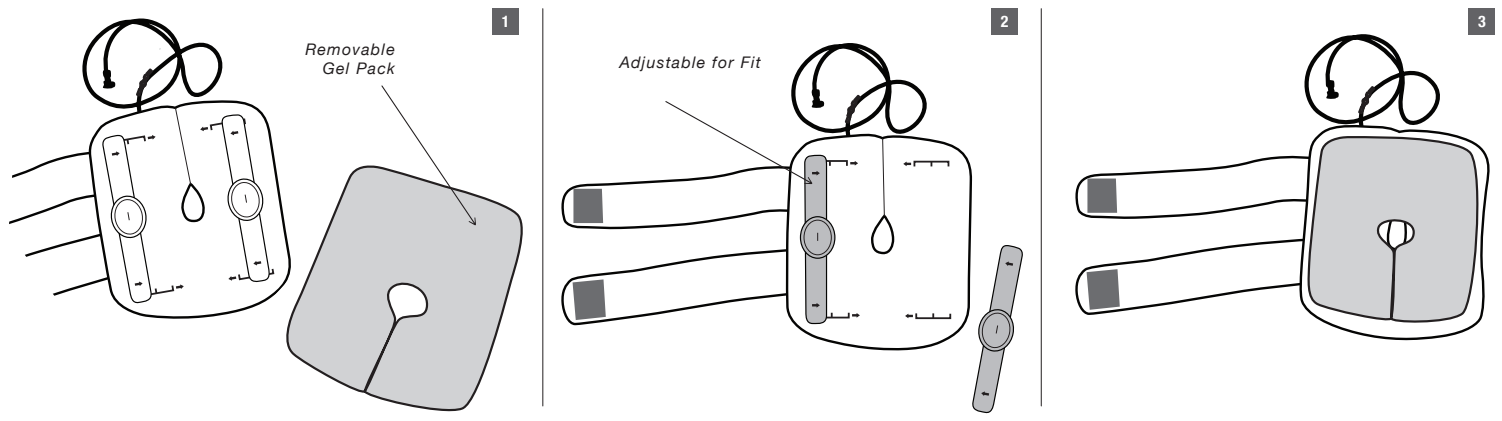For questions about this product call: **800.994.0464**

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